







codent

O AEOLUS6

AN EASY WAY TO ACHIEVE PERFECTION



WITH AEOLUS 6 GET MAXIMUM

Practice hygiene is becoming more important and is monitored increasingly. Ensure protection for your practice team and your patients without time-consuming.

The combination washer, thermo desinfector and lubricates up to six straight and contra-angle handpieces as well as turbines in 18 minutes at the push of a button with maximum hygienic safety in the AEOLUS 6.

CONFORMITY WITH STANDARDS

The washer thermo-desinfector process of the AEOLUS 6 meets the requirements of the European standard EN 15883. The cleaning and disinfection process is carried out in correspondence with the international standard for washer-disinfectors.



6 INSTRUMENTS EACH TIME

6_{*X*}





INTERNET VPN CONNECTIVITY FOR MOTORING SERVICE AT DISTANCE

AUTOMATIC RECORDING OF CYCLE REPORTS ON USB





THERMAL INTERNAL & EXTERNAL **PREWASHING, WASHING & DISINFECTION**



LUBRICATING WITH DYNAMIC **INTERNAL ROTATION PROCESS**



FAST DRYING PROCESS INTERNALLY AND EXTERNALLY





CITY WATER









LARGE TOUCH SCREEN COLOR, FRIENDLY AND EASY TO USE

ELECTRIC **CLOSING DOOR**

HIGH PERFORMANCE FAST & EASY TO USE FOR PERFECT HYGIENE

PERFECT HYGIENE OF YOUR **ROTATING INSTRUMENTS**

A simple press on the touch screen of the AEOLUS 6 triggers the automatic closing of the door and begins the cycle of cleaning, disinfection at high temperature of your instruments internally and externally in continuous rotation as well as the lubrication and maintenance of your instruments.

The entire program is fully automatic and reliable. All hygiene steps in the reprocessing of instruments, such as pre-washing, washing, disinfection, lubrication are carried out in a closed hygiene cycle in 18 minutes. Once the cycle is complete, the fully dried instruments will be available with a perfect hygiene result in compliance with the current EN 15883 standards.

FULL PROCESS OF AEOLUS6





Pre-washing with triples permanent rotation with low temperature 20°C to avoid protein coagulation





04 Drying the instrumentation

externaly and internaly

Internal lubrication of the rotative instruments with permanent internal rotation.






Washing at 45°C with chemical solution and triples permanent rotation





Thermal disinfection at 90° with triples permanent rotation

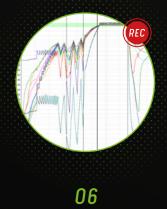

Recording the cycle

CODENT HYGIENIC PRODUCTS LINE, THE SOLUTION FOR THE FASTER AND HIGHEST LEVEL OF HYGIENE AND MAINTEANCE REQUIREMENTS

FULL EQUIPEMENT OF CODENT

Perfect, simple and fast hygiene that includes pre-washing, washing, disinfection, lubrication and sterilization for greater safety for you and your patients.

Codent offers you a complete hygiene solution to guarantee the longevity and performance of your handpieces, contra-angles and turbines, which will result in a better longevity



B-STANDARD STERILISATION CYCLE 22 MIN - 5.5 KG WITH SEALING BAGS B-PRION STERILISATION CYCLE 38 MIN - 5.5 KG WITH SEALING BAGS

BEYOND THE PERFORMANCE, AOLUS 6 IS ALSO ENHANCED BY A PURE AND INTELLIGENT DESIGN

Aeolus 6 notwithstanding to treat 6 instruments in the same time, equipped with an easy-to-use electric closing door allows to offer totally smooth surfaces, free from any ribs that could generate a bacterial nest. It becomes easy to clean for more hygiene and durability.



AEOLUS 6 STAINLESS STEEL

PRODUCTS & ACCESSORIES

AEOLUS 6 white	Ref. 100 000
AEOLUS 6 stainless steel	Ref. 100 100
Ao clean Solution 6 bottles of 1 liter	Ref. 100 500
Maintenance oil Oil, 6 bottles of 1 liter	Ref. 100 510
Ao clean Solution 1 bottles of 1 liter	Ref. 100 501
Maintenance oil Oil, 1 bottle of 1 Liter	Ref. 100 511
CO+ water purifying system	Ref. 100 520
Adaptator Etype	Ref. 100 001
Adaptor PTL	Ref. 100 002
Adaptor KV	Ref. 100 003
Adaptor SR	Ref. 100 004
Adaptor WH	Ref. 100 005
Adaptor BA	Ref. 100 006
	:

Ext Ca Ne Vo Su Ca Ca Со



AEOLUS 6 WHITE

TECHNICAL CHARACTERISTICS

xternal dimensions (H x W x D)	344 x 320 x 500 mm	
apacity	up to 6 instruments	
et weight	30 kg	
oltage CE	100 - 230 V / 50 - 60 Hz	
upply pressure	3,5 bars	
apacity of the oil tank	300ml	
apacity of the disinfection tank	300ml	
onformity with European Standard	EN-15883	

